



MODULAR
SERVICES COMPANY

USER MANUAL

≈ QUICK SHIP

MOBILE HEADWALL



OVERVIEW

The *Quick Ship* Mobile Headwall is a Class I, UL Recognized Medical Component manufactured by Modular Services Company (MSC). The configurable product is designed to be quickly deployed in an emergency for which a surge in healthcare capacity is expected and is able to add/expand electrical and medical gas services to a range of locations regardless of the infrastructure present in the space being used. For example, in a hospital setting, the Mobile Headwall can be configured to connect to existing medical gas outlets to rapidly convert an existing private room, so that additional patients may be treated. Conversely, if spaces without medical gas infrastructure are to be used, the Mobile Headwall provides the ability to supply the Medical Air and Oxygen outlets via compressed gas cylinders that mount directly to the unit. The Mobile Headwall can be relocated without removing the compressed gas cylinders for rapid deployment/redeployment. The Mobile Headwall can also be used in training environments.

Electric services are provided via hospital grade electrical receptacles which have LED indicators for a visual confirmation that the receptacles are live. Simply plug the hospital grade power cord plug into any standard 120-volt hospital grade power receptacle, normal or emergency. It can also be used in a non-healthcare environment.

UNPACKING AND INSPECTION

1. Upon receipt of Mobile Headwall units and prior to unpacking, inspect shipping containers for damage. Document any damage found and notify the carrier and Modular Services Company.
2. Carefully open each carton. Dispose of or recycle all excess packaging material.
3. Inspect unit for defects in materials or workmanship prior to assembly. It is the responsibility of the customer to report any damage or deficiencies to Modular Services immediately upon discovery.

ASSEMBLY

The Mobile Headwall ships completely assembled. No additional assembly is required.

SET UP AND FEATURES

IMPORTANT: Review this User Manual and all safety labels attached to the unit before proceeding.

UNIT OPERATION

1. Lock all three (3) braking casters before placing the Mobile Headwall into service.



CAUTION: Although the Mobile Headwall is safe to transport on/across uneven/unlevel surfaces, it is not intended for end use on uneven/unlevel surfaces.

2. To energize the Mobile Headwall, Plug the power cord plug into a standard normal or emergency 120 volt wall power outlet. The unit will now provide power from the LED indicating hospital-grade power outlets. To discontinue use, unplug the power cord plug from the power source.



RATING: 120V. 12 A. 60Hz Max.

3. To connect Mobile Headwall to medical gas or vacuum, attach the gas hoses to existing medical gas or vacuum outlets or to medical gas cylinders or vacuum pump.



CAUTION: The Mobile Headwall is designed for the storage and transportation of M-60 and M-122 High Pressure Medical Gas Cylinders only. Ensure Cylinders are securely fastened to the Mobile Headwall with the provided straps.

4. The Mobile Headwall is not intended for use during patient transportation, only as a stationary device. If transportation is required, ensure the medical gas hoses and power cord are wrapped up on the sides of the headwall.

IMPORTANT: Ensure high pressure medical gas cylinders have been prepared for transportation by removing regulators and installing safety caps. Always maintain contact with the unit to avoid losing control of the unit's motion.

CARE AND MAINTENANCE

Maintenance

- For parts or repairs, contact your sales representative or Modular Services Company.

Cleaning

- Clean the Mobile Headwall with mild detergent and warm water or as directed in your facility cleaning policy. Cleaning after each patient is recommended.
- Avoid buildup of excessive moisture.
- Disinfect as required, according to location requirements.
- To remove difficult spots or stains, use standard household cleansers and a soft-bristled brush.

WARRANTY

Modular Services Company warrants that all equipment assemblies shall be free from defects in material and workmanship for a period of 12 months from date of the owner's acceptance to the installing contractor or the date the equipment is put into service, whichever comes first. Warranty excludes electric lamps and/or any material not furnished by Modular Services. Warranty does not cover damage due to improper installation and/or abuse.

It is the responsibility of the customer to report any noted product deficiencies to Modular Services immediately upon discovery. It is the responsibility of Modular Services to expediently resolve the discrepancy. Any modification made to the product without the written authorization from Modular Services will void this warranty.

Also, in the event product modifications or repairs are made without the written consent of Modular Services, Modular Services shall not be liable for any cost associated with the modification or repair.

There are no warranties of fitness which extend beyond the description on the face hereof.



500 E. Britton Rd. • Oklahoma City, OK 73114

Tel: 800.687.0938 • Fax: 405.528.0368

www.modularservices.com

info@modularservices.com